



# GENTHERM

## MAXI-THERM® LITE STAFF VEST

**REF 300**



Fig. 1 Attach velcro on each side

### WARNINGS

1. Leaky vests or hoses should never be used. Water leaks present a risk of infection and should be handled accordingly.
2. Do not place sharp objects in contact with the Maxi-Therm® Lite Staff Vest as punctures may occur. Do not use pins to secure the Vest.
3. The Maxi-Therm® Lite Staff Vest is for single O.R. professional use only. Reuse may result in biocontamination or cross-infection.

### PRECAUTIONS

1. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
2. The Maxi-Therm® Lite Staff Vest is intended to be used on O.R. professionals. The Maxi-Therm® Lite Staff Vest is not intended to be used on patients.
3. Discontinue use if the vest becomes uncomfortable, starts to cause excessive sweating, chills, shivering or skin irritation.
4. Use of unapproved equipment or hoses is not recommended. The Maxi-Therm® Lite Staff Vest has been designed and tested for use with Gentherm thermal regulating systems only.

### INDICATIONS FOR USE

The Maxi-Therm® Lite Staff Vest is intended to be used by trained healthcare professionals as a part of Gentherm's thermal regulating systems, specifically the Blanketrol® II, Blanketrol® III, or Norm-O-Temp® to lower or raise body temperature and/or maintain a desired body temperature through conductive heat transfer.

### CONTRAINDICATIONS

None known.

### SET UP

1. Remove the Maxi-Therm® Lite Staff Vest from packaging. Verify product integrity prior to each use.
2. The hose should be placed in the back and the "V" neck in the front.
3. Slide the Maxi-Therm® Lite Staff Vest over the head making sure that the label, "This side to Body" is facing the body.
4. Fasten each side by adjusting the Velcro for proper fit (see Fig. 1, above).
5. The Maxi-Therm® Lite Staff Vest may be worn on top of surgical scrubs and under sterile gown aprons.
6. Attach the hose to the cooling and/or heating unit.

### OPERATING INSTRUCTIONS

1. Follow the Gentherm thermal regulating system operation manual for operating instructions.

### CLEANING

1. The Maxi-Therm® Lite Staff Vest is not sterile and is intended for single O.R. professional use. Do not reprocess or sterilize. Discard the Maxi-Therm® Lite Staff Vest according to hospital/institution protocol after each use.

### STORAGE

1. Store unused Maxi-Therm® Lite Staff Vest in cool, dry environment, free from items that may puncture Maxi-Therm® Lite Staff Vest.

Gentherm Medical, LLC  
12011 Mosteller Road  
Cincinnati, Ohio, USA 45241  
Tel.: 1-800-989-7373  
[www.gentherm.com](http://www.gentherm.com)

56423-J  
(01/2022)