DA
Anvend kun Blanketrol®, Electri-Cool® II, Norm-O-Temp® eller Hemotherm.

FORBEREDELSE:

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SETUP INSTRUCTIONS:

1 Open pad on table. Connect to hyper/hypothermi a device.	2 Position patient on pad then turn on hyper/hypothermia device.	3 Fit pad to forehead.	4 Fit pad to neck.	5 Fit pad to top of head.

Indications:

For patient temperature management by trained caregivers only.

Contraindications:

Do not leave patient with poor perfusion unmonitored during extended procedures. Patients at greatest risk are those unconscious, on extended therapy, diabetics, children, and totally immobile patients. Presence of water or gel on the head wrap with the unit not operating may cool the patient.

Cautions:

- Do not apply wrap too tightly. Use care when fitting to forehead and neck. Be sure to activate unit and leave on before fitting to patient. The pad will become tighter on patient if it is filled after being fitted.
- Be sure to check position of pad to ensure that the ears are not in contact with the pad.
- Check skin conditions frequently, due to individual differences in sensitivity and susceptibility to injury from cold. Patients at greatest risk are those unconscious or incapacitated, persons on prolonged therapy, diabetics, children, persons with insensitive skin areas or poor circulation.

- Federal (USA) Law restricts this device to sale by or on the order of a physician.

Warnings:

- Do not use on children under the age of 12 without doctor's supervision.
- Do not cover ears with this product.
- Do not use on an unconscious person without doctor's supervision.

- Use of this product may cause the following:
 - Drowsiness or fatigue
 - Uncontrolled sleepiness
 - Slowed reactions
 - Initial cold discomfort sensations
 - Skin irritation or redness
 - Blurred vision
 - Dizziness
 - Thermal injury
- Do not wrap tightly enough to restrict blood flow.
- Do not use pins or other sharp objects to secure wrap or hoses. Do not allow wrap or hoses to come in contact with any sharp objects.
- Do not use the device distal to arterial cross clamping. Non-observance can lead to thermal injury.
- Hazard of patient injury. Thermal injury may occur if heating/cooling therapy is applied to ischemic limbs.
- This disposable device is intended for single patient use only. Reuse may result in biocontamination or cross-infection.

	GENTHERM		
	Gentherm Medical, LLC 12011 Mosteller Road Cincinnati, Ohio, USA 45241 Tel: 1-800-989-7373 www.gentherm.com	0344	CEpartner4U, BV Esdoornlaan 13 3951 DB Maarn The Netherlands www.CEpartner4U.com

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