# PlastiPad®

## Narrow Adult

Reusable Hyper-Hypothermia Blanket  
Thermal Regulating System

### INDICATIONS FOR USE

The PlastiPad® reusable hyper-hypothermia blanket is intended to be used by trained healthcare professionals as part of Gentherm's thermal regulating systems, specifically the Blanketrol® II, Blanketrol® III, Hemotherm®, Norm-O-Temp®, or Micro-Temp LT® to lower or to raise a patient's temperature and/or maintain a desired patient temperature through conductive heat transfer.

### CONTRAINDICATIONS

None known.

- Unfold blanket on bed, stretcher or operating room (OR) table with hose routed without kinks towards the unit. Check blanket before each use for product integrity.
- Check the equipment reservoir to ensure it is full of sterile water or that has been passed through a filter of less than or equal to 0.22 microns.
- Connect male coupling and female coupling from the Cat#286 hose to unit. Make sure the "click" as they lock together.
- Attach blanket to the Cat#286 hose by inserting the blanket connectors into the hose connectors. Connectors should "click" as they lock together.
- Turn unit ON and select desired temperature setting.
- Place a single sheet between the patient's skin and blanket. The blanket may be filled either before or after patient is placed on blanket. Blanket may be under or over patient as desired.
- After filling and positioning, make sure blanket and hose are free of kinks that may restrict water flow.

### OPERATING INSTRUCTIONS

- Follow the Gentherm thermal regulating system operation manual for operating instructions.
- To discontinue therapy: turn the regulating system unit off. The water in the blanket will automatically drain back into the unit. Allow it to fully drain before disconnecting the blanket from unit or removing from bed, stretcher, or table.

### CLEANING

Maintenance may be performed by nursing personnel or by other trained persons. The following actions are considered maintenance: Replacing hoses, cleaning hoses, and blankets.

- Clean blanket with a mild soap and water solution. Disinfect according to hospital/institution protocol. Gentherm does not recommend sterilization. Do not autoclave.
- To store blanket, gently fold corner containing the tubing towards the center of the blanket. Roll or fold blanket loosely avoiding sharp creases.
- Store blanket in cool, dry environment, free from items that may puncture blanket.

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