# PlastiPad®

## Pediatric

Reusable Hyper-Hypothermia Blanket  
Thermal Regulating System

**INDICATIONS FOR USE**  
The PlastiPad® reusable hyper-hypothermia blanket is intended to be used by trained healthcare professionals as part of Gentherm's thermal regulating systems, specifically the Blanketrol® II, Blanketrol® III, Hemotherm® or Norm-O-Temp® to lower or raise a patient's temperature and/or maintain a desired patient temperature through conductive heat transfer.

**CONTRAINDICATIONS**

None known.

- SET UP**  
1. Unfold blanket on bed, stretcher or operating room (OR) table with hose routed without kinks towards the unit.  
2. Check the equipment reservoir to ensure it is full of sterile water or water that has been passed through a filter of less than or equal to 0.22 microns.  
3. Connect male coupling and female coupling from the Cat#286 hose to unit. Make sure the "click" as they lock together.  
4. Attach blanket to the Cat#286 hose by inserting blanket connectors into the hose connectors. Connectors should "click" as they lock together.  
5. Turn unit ON and select desired temperature setting.  
6. Place a single sheet between the patient's skin and blanket. The blanket may be filled either before or after patient is placed on blanket. Blanket may be under or over patient as desired.  
7. After filling and positioning, make sure blanket and hose are free of kinks that may restrict water flow.

- OPERATING INSTRUCTIONS**  
1. Follow the Gentherm thermal regulating system operation manual for operating instructions.  
2. To discontinue therapy: turn the regulating system unit off. The water in the blanket will automatically drain back into the unit. Allow it to fully drain before disconnecting the blanket from unit or removing from bed, stretcher, or table.

- CLEANING**  
Maintenance may be performed by nursing personnel or by other trained persons. The following actions are considered maintenance:  
1. Clean blanket with a mild soap and water solution.  
2. Do not place bean bags or other firm positioning devices under the blankets/pads. Skin damage may occur.  
3. In surgery, heating may affect toxicity of certain prep solutions which may cause skin injury when remaining on patients' skin during application of prolonged heat.  
4. Do not use sharp objects in contact with the PlastiPad® blanket as punctures may occur. Do not use pins to secure PlastiPad® blanket.

- STORAGE**  
1. To store blanket, gently fold corner containing the tubing towards the center of the blanket. Roll or fold blanket loosely avoiding sharp creases.  
2. Store blanket in cool, dry environment, free from items that may puncture blanket.

- DISPOSAL**  
1. Do not dispose of PlastiPad® in regular trash. Disposal must be in accordance with local regulations.

- GENTHERM**  
Gentherm Medical, LLC  
12011 Mosteller Road  
Cincinnati, Ohio, USA 45241  
Tel: 1-800-989-7373  
www.gentherm.com

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